EN Mediblink ColdHot Pack

INSTRUCTIONS FOR USE

INTENDED USE

Providing cold/heat therapy for different affected areas.

Cold pack: physical therapy for traumatic hemorrhage, heatstroke, itch, headache, burns, fatigue, sprains, strains, blows and bruises, toothache and other types of pain.

Hot pack: for alleviating rheumatic pain, neural pain, sciatica, vertebral pain, reducing mucous membrane scarring after surgery, for arthritis, pain caused by strain, swelling and traumatic recovery, cramps, back pain, stiff neck, lumbago, menstrual pain, for keeping warm at low temperatures.

METHOD OF USE

COLD THERAPY:

Store the pack in the freezer for at least 2 hours before using, or keep one pack in the refrigerator at all times for immediate use.

Wrap the cold pack in a cover, towel or a piece of cloth and apply to the affected area (only for ColdHot Pack with cover [REF](#) M127, M128 and M129).

HEAT THERAPY:

A) Heating the pack in a microwave:

Put the pack in the microwave, set power level to high. Since microwave ovens vary, you may need to microwave the pack for a shorter or longer time to reach the desired temperature. Adjust the heating time accordingly. Wrap the warm pack in a cover, towel or a piece of cloth and apply to the affected area (only for ColdHot Pack with cover [REF](#) M127, M128 and M129).

max 1.000 W / 25 sec
max 900 W / 30 sec
max 700 W / 35 sec

B) Heating the pack in hot water:

- Heat water in a pot until it begins to boil.
- Remove the pot from the heat source.
- Immerse the pack in hot water for 8 to 10 minutes.
- Remove the pack from water.
- Check the temperature. If the pack is too hot, allow it to cool slightly.
- When the desired temperature is reached, wrap the pack in a cover, towel or piece of cloth (only for ColdHot Pack with cover [REF](#) M127, M128 and M129).
- Apply to the affected area.

max 60 °C / 10 min

THE COLDHOT PACKS RANGE

Product Ref. number / Name	Advantage
REF M120 / ColdHot Pack with Beads S 10 x 10 cm M120	Pack with beads which can be put directly on the affected area , without wrapping the pack in a cover, towel or a piece of cloth.
REF M121 / ColdHot Pack with Beads M 12 x 29 cm M121	
REF M122 / ColdHot Pack with Beads L 20 x 30 cm M122	
REF M123 / ColdHot Pack with Beads Breast 17 x 17 cm 2x M123	
REF M124 / ColdHot Pack with Beads Frog 10,5 x 11,5 cm M124	
REF M125 / ColdHot Pack with Beads Ladybird 11,4 x 11,5 cm M125	
REF M126 / ColdHot Pack with Beads with belt XL 20,5 x 38,5 cm M126	Wrap the ColdHot pack in a cover, towel or a piece of cloth before applying to bare skin (only for ColdHot Pack with cover REF M127, M128 and M129).
REF M127 / ColdHot Pack with cover S 11 x 12 cm M127	
REF M128 / ColdHot Pack with cover M 11 x 26 cm M128	
REF M129 / ColdHot Pack with cover L 20 x 30 cm M129	

CLEANING

Wash with warm water and use washing up liquid if necessary. Wipe the pack dry with a clean towel.

CONTRAINDICATIONS

The ColdHot pack is suitable for everyone except persons suffering from the following diseases:

For cold pack:

- Raynauds phenomenon – A condition affecting small blood vessels in the extremities. Exposure to cold may cause the symptoms to reoccur.
- Cold hypersensitivity – There are many forms of cold hypersensitivity, including:
 - Cold urticaria – Also known as hives. A release of histamine during re-warming after a period of cooling, causing red, itchy welts on the skin.
 - Cold erythema – A rash, characterised by redness and itching, caused by exposure to cold. Can cause severe pain and muscular spasms.
 - Cold hemoglobinuria – When red blood cells break down so quickly that some haemoglobinuria cannot combine with blood proteins.
- Anesthesia – When the patient experiences numbness or altered sensitivity, cold therapy should not be used as the patient cannot feel pain and the degree of cooling.
- Heart disease – such as arrhythmia, angina or CHD.
- High blood pressure – Vasoconstriction could increase blood pressure.

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- Superficial nerves – Cold therapy should not be used on an area where nerves pass close to the surface.
- Healing wounds – Cold therapy should be used over a new wound.

For hot pack:

- Acute abdominal pain – Do not use hot therapy before obtaining a diagnosis.
- Purulent infection – Hyperthermia can dilate blood vessels and lead to bacteria and toxins entering the blood circulation, which may cause a spread of inflammation and severe intracranial infection and sepsis.
- Visceral hemorrhage – Because hyperthermia can dilate local blood vessels, it will increase the blood flow in organs and vascular permeability, thereby increasing bleeding.
- Preliminary stage of soft tissue bruise or sprain – There is an acknowledged risk of subcutaneous hemorrhage and edema from hyperthermia.
- Skin eczema/allergic dermatitis – May cause the symptoms to worsen.
- Healing wounds – Hot therapy should not be used over a new wound.

For both cold and hot packs:

- Impaired sensation.
- Cognitive or communication deficits that preclude reporting of pain, including sedation, mental issues or dementia.
- Irradiated skin.
- Fragile skin (such as found in the elderly, small infants, or patients on steroid therapy).

⚠ WARNINGS

- For external use only.
- Do not use for purposes other than intended. Persons with known or suspected circulatory problems should not use a cold or hot pack except under direct supervision of a doctor.
- Never sit, lie or lean against the cold/hot pack.
- Do not fall asleep using the cold/hot pack.
- Do not use on infants.
- Use only under adult supervision.
- Overheating or prolonged application may cause injury.
- Do not open the cold/hot pack.
- Do not puncture pack and/or ingest contents.
- Do not apply to sensitive skin, insensitive skin or open wounds. For extreme, prolonged or chronic pain, consult a doctor.
- Stop using the pack if damaged.

Any serious incident that has occurred in relation to the device should be reported to the Manufacturer and the Competent Authority of the Member State in which the user and/or patient is established.

Storage conditions: Store in a cool place.

Shelf Life: 3 years

SYMBOLS

Symbol	Meaning
	Before use, read the instructions.
	Use by
	Manufacturer's name and address
	EU Representative's name and address
	Warning
	Do not use if package is damaged
	Reference number
	Batch number*
	Medical device
	Unique Device Identification system
	Number of products in one packaging
	Paper
	Plastic

* The production date can be read from Batch number (YYYYMM); first four digits represent the Year and last two digits Month of production. Example: LOT 202503 = March 2025

Manufactured for (Importer for EU & Distributor):
Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje, Slovenija,
www.mediblink.com

Intco Medical (HK) Co., Limited
Flat/RM 19C, Lockhart Centre, 301-307 Lockhart Road,
Wan Chai, Hong Kong, China

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MedNet EC-REP GmbH