
EN Mediblink Gallium thermometer with magnifying glass Vision M371

DESCRIPTION OF THERMOMETER COMPONENTS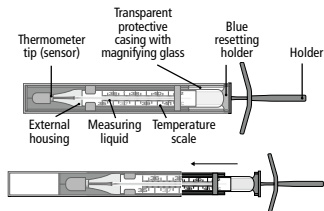

SPECIFICATION

Model / REF: CR.W00 / M371 • Minimum measuring range: 35.5°C to 42°C, minimum interval is 0.10°C • Accuracy: +0.10°C and -0.15°C, it is recommended to measure at 37°C and 41°C • Storage conditions: Temperatures between -10°C and 42°C • Operating conditions: Temperatures from 20°C to 30°C • Standard: EN12470:2000+A1:2009

⚠ WARNINGS

1. Warn patient against chewing, speaking and playing when thermometer is below the tongue of patient.
2. Need to let sick child to lie quietly and help him/her to hold another end of the thermometer for rectal use in case it is inserted too deep, or it slips from the anus.
3. Avoid having a bath, eating or smoking half an hour before measuring body temperature.
4. Thermometer is not designed for special applications (e.g., thermometers for premature babies, ovulation thermometers).
5. Always disinfect thermometer after use.
6. Do not bend or chew the tip of the thermometer.
7. Do not use thermometer with damaged external housing (danger of injury).
8. The measuring fluid (a mixture of gallium, tin and indium, hereinafter: "fluid") must not come into contact with precious or light metals as it reacts to them.
9. Care should be taken to avoid any potential risk or damage during measuring.
10. Children aged 10 and under who need to use a thermometer to measure their temperature can only use it with the assistance of an adult.
11. Use of this thermometer is not intended as substitute for visiting your doctor. Persistent fever, especially in children must be treated by a health professional.
12. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the Com-

petent Authority of the Member State in which the user and/or patient is established.

INTENDED USE

1. Intended purpose: For measuring the temperature of human.
2. Expected operating environment: Minimum measuring temperature range should be from 35.5°C to 42°C, and the environment temperature should be from 20°C to 30°C; if the temperature for storage is less than 20°C, please put the thermometer in the temperature 20°C to 30°C for at least 1–2 hours or 30°C to 35°C for ten minutes before use or test.
3. Method of use:
 - a) Remove the transparent protective casing with the magnifying glass.
 - b) Pull the thermometer out of the blue resetting holder.
 - c) Perform the measurement (see section 4).
 - d) Reinsert the thermometer into the blue resetting holder.
 - e) Reattach the transparent protective casing with the magnifying glass.
 - f) Read the measured temperature.
 - g) **IMPORTANT: To reset the temperature reading, first remove the transparent protective casing with the magnifying glass.** Then firmly hold the tail end of the blue resetting holder (holder) of the thermometer between your thumb and forefinger and carefully swing the thermometer several times until the liquid column falls back down (image).
 - h) Reattach the protective casing for storage.
4. Operation procedures:
 - a) The liquid column shall fall below the lowest numbered scale mark before use, then clean and disinfect the thermometer in medical alcohol or put on a sterilized plastic tube.
 - b) Be careful to put it below the tongue of patients, if measuring oral position and tell them to close their mouth tightly.
 - c) For rectal use of thermometer apply some lubricator before inserting it in the anus.
 - d) Be careful to put it below the armpit of patients, if measuring armpit position and tell them to press the hand against the chest.
 - e) Clean the glass thermometer after use.

Please keep the thermometer measuring for about 4 minutes before reading the number on the thermometer scale.

CLEANING INSTRUCTIONS

The thermometer has hermetically sealed hygienic glass housing and can be disinfected without restrictions.

- ⚠ The temperature of disinfectants and cleaning agents shall not exceed 42°C. Clean the glass thermometer with a medical alcohol (such as 75% Ethanol Alcohol or 70% Isopropyl Alcohol) before and after use. Disinfect the thermometer with a cotton pad or cloth soaked in alcohol or immersed in a disinfectant solution (immersion time must be in accordance with its manufacturer's instructions).

WASTE DISPOSAL

If the thermometer is shattered, spilled liquid may be removed by a brush, or a cloth, soaked in soapy water. The spilled fluid is non-toxic. Broken parts may be disposed as household waste.

STORAGE REQUIREMENTS

The temperature should keep between -10°C and 42°C when stored or transported.

SHELF LIFE

The service life of the mercury-free thermometer shall be considered expired in the following cases: The thermometer shows visible damage (e.g., cracks, breakage, leakage). The metal liquid column in the capillary is interrupted or separated. The metal liquid column descends or reverses direction without external influence. The metal liquid column is immobilized and cannot be moved by gentle manual manipulation. If any of the above conditions are observed, discontinue use of the thermometer immediately.

WARRANTY

Product: Mediblink Gallium thermometer with magnifying glass Vision M371

Manufactured for (Importer for EU & Distributor):

Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje, Slovenia;
info@mediblink.com; www.mediblink.com

Seller's name, address, signature and stamp*:

Date of extradition / sales*:

*If the invoice is accompanied by this warranty, and if all above information can be seen from the invoice, it is not necessary to fill in this field.

WARRANTY TERMS

Dear customers!

The warranty period is **10 years** and starts from the day of product purchase. In case of product claim, you have to show the invoice. We kindly ask you to save the invoice!

Unfortunately, wrong handling with the device is a reason for 95% of customer complains. You can easily avoid any problem, by getting useful information provided by our special service department. To reach our service department, you can call or send e-mail to Mediblink local distributor.

Before sending the product back to retailer, we kindly ask you to call our service department, to get help about how to use the device to save you with unneeded trips.

The manufacturer guarantees free elimination of all imperfections due to defects in material or manufacturing procedure by repairing or replacing the product. In case that the product can not be repaired or replaced, the customer will get the money refund. The guarantee is not valid in case of the force majeure, accidents or unexpected events (such as lightning, water, fire

etc.), incorrect use or incorrect transport, non-compliance with safety and maintaining regulations or in case of unprofessional product intervention.

Traces of every day product usage (scratches, abrasions) are not subject to claim. The warranty does not eliminate the customer rights, which originate from seller responsibility for product flaws. By accepting the claimed product by the service department, the service department does not take responsibility for loss of saved data or settings on the product. All product repairs, which are performed out of product warranty period, have to be paid by customer by prior notice. The manufacturer guarantees the product quality and flawless product operation in the warranty period, which starts with the day of product purchase. If the product can not be repaired in 45 days, the product will be replaced with a new one. In case that the product can not be replaced, the money will be refunded to the customer. In case of product claim, call or send e-mail to Mediblink local distributor.

LIST OF SYMBOLS

	Consult instructions for use or consult electronic instructions for use http://www.mediblink.com/f/m371.pdf		
	Caution		Unique Device Identifier
	Reference number		Medical device
	Batch number*		Manufacturer
	Authorized Representative in EU		The product does not contain mercury
	Temperature limit		Fragile: handle with care

*The production date can be read from LOT number [YYYYMM]; first four digits represent the Year and last two digits Month of production. Example: LOT 202503 = March 2025

Wuxi Medical Instrument Factory Co., Ltd.
No.43 Xixin Road, Zhangjing, Xibei Town,
Wuxi City, 214194 Jiangsu, P.R. China
Email: wxmedical@vip.126.com

Lotus NL B.V., Koningin Julianaplein 10,
1e Verd, 2595AA, The Hague, Netherlands
Email: info@lotusnl.com

CE
0197

Manufactured for (Importer for EU & distributor):

Mediblink d.o.o., Gubčeva cesta 19, 8210 Trebnje, Slovenia
Contact: info@mediblink.com / www.mediblink.com

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Lotus NL B.V., Koninjing Julianaplein 10,